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Phosphate buffer PH 7.4 80 mmol/l
NaCl 150 mmol/l

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Glycine buffer PH 8.2 20 mmol/l
NaCl 150 mmol/l
Latex particles coated with streptolysin O

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BT 3500/1500/3000/2000/1000

BT 3500/1500		
ASO		
Primary Parameters	method	Fixed Time
	Kind of process	linear
	filter	630/-
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	180
	Reagent2	18
	SAMPLE	
	sample vol	2
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
	Reading time(sec)	300
Incubation time(sec)	180/10	
Check parameters	Reagent limit(M ABS)	2000
	Curve acceptance	100
	Test limit(conc)	400
	Initial ABS(M ABS)	0
	Final ABS(M ABS)	3000
	Max ABS delta (M ABS)	2000
	Chec prozone	inactive
	NORMAL RANGE	
	Male	0 - 200
	female	0 - 200
	Child	0 - 150
	Re-run hyperactive	inactive
	Re-run pathological	inactive
Secondary	Unit serum	IU/ml
	Num of needle washes	1/1
	Num of cuvette wash	2
	Additional wash	inactive
	Only needle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyrun
	Decimals	0

BT 3000/2000/1000		
ASO		
Primary Parameters	method	Fixed Time
	Kind of process	linear
	filter	630/-
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	300
	Reagent2	30
	SAMPLE	
	sample vol	3
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
	Reading time(sec)	300
Incubation time(sec)	180/10	
Check parameters	Reagent limit(M ABS)	2000
	Curve acceptance	100
	Test limit(conc)	400
	Initial ABS(M ABS)	0
	Final ABS(M ABS)	3000
	Max ABS delta (M ABS)	2000
	Chec prozone	inactive
	NORMAL RANGE	
	Male	0 - 200
	female	0 - 200
	Child	0 - 150
	Re-run hyperactive	inactive
	Re-run pathological	inactive
Secondary	Unit serum	IU/ml
	Num of needle washes	1/1
	Num of cuvette wash	2
	Additional wash	inactive
	Only needle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyrun
	Decimals	0

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