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TRIS	PH 8.0	100 mmol/l
NaCl		320 mmol/l
Anti-human C3c antibody (goat) with stabilizers		

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1. Thomas L. Clinical Laboratory Diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 794-806.
2. Johnson AM, Rohlfes EM, Silverman LM. Proteins. In: Burtis CA, Ashwood ER. editors. Tietz textbook of clinical chemistry. 3rd ed. Philadelphia: W. B. Saunders Company; 1999. p. 507-12.
3. Dati F, Schumann G, Thomas L, Aguzzi F, Baudner S, Bienvenu J et al. Consensus of a group of professional societies and diagnostic companies on guidelines for interim reference ranges for 14 proteins in serum based on the standardization against the IFCC/BCR/CAP reference material (CRM 470). Eur J Clin Chem Clin Biochem 1996;34:517-20.

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BT 3500/1500/3000/2000/1000

BT 3500/1500		
C3		
Primary Parameters	method	Sample Blank(A)
	Kind of process	Multi ponit
	filter	340/-
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	180
	Reagent2	30
	SAMPLE	
	sample vol	18
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
	Reading time(sec)	10
	Incubation time(sec)	300/300
Check parameters	Reagent limit(M ABS)	2000
	Curve acceptance	100
	Test limit(conc)	500
	Initial ABS(M ABS)	0
	Final ABS(M ABS)	3000
	Max ABS delta (M ABS)	2000
	Chec prozone	inactive
	NORMAL RANGE	
	Male	90 - 180
	female	90 - 180
	Child	90 - 180
	Re-run hyperactive	inactive
	Re-run phatological	inactive
Secondary	Unit serum	mg/dl
	Num of needle washes	1/1
	Num of cuvette wash	2
	Additional wash	inactive
	Only neadle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyrun
	Decimals	0

BT 3000/2000/1000		
C3		
Primary Parameters	method	Sample Blank(A)
	Kind of process	Multipoint
	filter	340/-
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	300
	Reagent2	50
	SAMPLE	
	sample vol	30
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
	Reading time(sec)	10
	Incubation time(sec)	300/300
Check parameters	Reagent limit(M ABS)	2000
	Curve acceptance	100
	Test limit(conc)	500
	Initial ABS(M ABS)	0
	Final ABS(M ABS)	3000
	Max ABS delta (M ABS)	2000
	Chec prozone	inactive
	NORMAL RANGE	
	Male	90 - 180
	female	90 - 180
	Child	90 - 180
	Re-run hyperactive	inactive
	Re-run phatological	inactive
Secondary	Unit serum	mg/dl
	Num of needle washes	1/1
	Num of cuvette wash	2
	Additional wash	inactive
	Only neadle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyrun
	Decimals	0

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