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1. Stein W. Creatine Kinase (total activity), creatine kinase isoenzymes and variants. In: Thomas L, ed. Clinical laboratory diagnostics. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p.71-80.
2. Moss DW, Henderson AR. Clinical enzymology. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of clinical chemistry. 3<sup>rd</sup> ed. Philadelphia: W.B Saunders company; 1999. p.617-721.
3. Recommendations of the German Society for Clinical Chemistry. Standardization of methods for the estimation of enzyme activities in biological fluids: Standard method for the determination of creatine kinase activity. J clin chem. Clin Biochem 1977; 15:255-60.
4. Lorenztz K, Rohle G, Siekmann L. Introduction of new standard methods 1994 for the determination of catalytic enzyme concentrations at 37 c. DG Klinische chemie Mitteilungen 1995; 26:290-3.

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BT 3500/1500/3000/2000/1000

BT 3500/1500		
CPK		
Primary Parameters	method	kinetic
	Kind of process	linear
	filter	340/405
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	160
	Reagent2	40
	SAMPLE	
	sample vol	12
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
Reading time(sec)	120	
Incubation time(sec)	300/120	
Check parameters	Reagent limit(M ABS)	800
	Curve acceptance	100
	Test limit(conc)	800
	Initial ABS(M ABS)	300
	Final ABS(M ABS)	1500
	Max ABS delta (M ABS)	350
	Chec prozone	inactive
	NORMAL RANGE	
	Male	24 - 195
	female	24 - 170
	Child	24 - 330
	Re-run hyperactive	inactive
	Re-run phatological	inactive
Secondary	Unit serum	U/L
	Num of needle washes	1/1
	Num of cuvette wash	2
	Additional wash	inactive
	Only neadle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyday
	Decimals	0

BT 3000/2000/1000		
CPK		
Primary Parameters	Method	kinetic
	Kind of process	linear
	Filter	340/405
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	240
	Reagent2	60
	SAMPLE	
	sample vol	14
	Pre dilution	1
	Dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
Reading time(sec)	120	
Incubation time(sec)	300/120	
Check parameters	Reagent limit(M ABS)	800
	Curve acceptance	100
	Test limit(conc)	800
	Initial ABS(M ABS)	300
	Final ABS(M ABS)	1500
	Max ABS delta (M ABS)	350
	Chec prozone	inactive
	NORMAL RANGE	
	Male	24 - 195
	Female	24 - 170
	Child	24 - 330
	Re-run hyperactive	inactive
	Re-run phatological	inactive
Secondary	Unit serum	U/L
	Num of needle washes	1/1
	Num of cuvette wash	2
	Additional wash	inactive
	Only neadle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyday
	Decimals	0

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