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1. Wick M, Pingerra W, Lehmann P. Iron metabolism: diagnosis and therapy of anemias. 3rd ed. Vienna, New York: Springer Verlag, 1996.
2. Fairbanks VF, Klee GG. Biochemical aspects of hematology. In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1999. p. 1642-1710.
3. Dati F, Schumann G, Thomas L, Aguzzi F, Baudner S, Bienvenu J et al. Consensus of a group of professional societies and diagnostic companies on guidelines for interim reference ranges for 14 proteins in serum based on the standardization against the IFCC/BCR/CAP reference material (CRM 470). Eur J Clin Chem Clin Biochem 1996;34:517-20.

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BT 3500/1500/3000/2000/1000

BT 3500/1500		
TRANSFERIN		
Primary Parameters	method	Sample Blank(A)
	Kind of process	Multi ponit
	filter	340/-
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	200
	Reagent2	20
	SAMPLE	
	sample vol	10
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
	Reading time(sec)	10
Incubation time(sec)	300/300	
Check parameters	Reagent limit(M ABS)	2000
	Curve acceptance	100
	Test limit(conc)	800
	Initial ABS(M ABS)	0
	Final ABS(M ABS)	3000
	Max ABS delta (M ABS)	2000
	Chec prozone	inactive
	NORMAL RANGE	
	Male	250 - 450
	female	250 - 450
	Child	250 - 450
	Re-run hypeactive	inactive
	Re-run phatological	inactive
Secondary	Unit serum	mg/dl
	Num of needle washes	2/2
	Num of cuvette wash	2
	Additional wash	inactive
	Only needle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyday
	Decimals	0

BT 3000/2000/1000		
TRANSFERIN		
Primary Parameters	method	Sample Blank(A)
	Kind of process	Multipoint
	filter	340/-
	Reaction direction	increase
	REAGENT	
	Num of reagent	2
	Reagent1	300
	Reagent2	30
	SAMPLE	
	sample vol	12
	Pre dilution	1
	dilution	5
	TIME	
	Sample starter	inactive
	Delay time(sec)	0
	Reading time(sec)	10
Incubation time(sec)	300/300	
Check parameters	Reagent limit(M ABS)	2000
	Curve acceptance	100
	Test limit(conc)	800
	Initial ABS(M ABS)	0
	Final ABS(M ABS)	3000
	Max ABS delta (M ABS)	2000
	Chec prozone	inactive
	NORMAL RANGE	
	Male	250 - 450
	female	250 - 450
	Child	250 - 450
	Re-run hyperactive	inactive
	Re-run phatological	inactive
Secondary	Unit serum	mg/dl
	Num of needle washes	2/2
	Num of cuvette wash	2
	Additional wash	inactive
	Only needle	
	Only wash	
	Instrument factor	1
	Shift:	0
	Reagent blank	everyday
	Decimals	0

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